



# User Guide

## qube**AV**pro LAN **light**



Date: 10/2022



<b>General</b>	3
Introduction	3
Sustainable quality management	3
Important safety measures	4
<b>qubeAVpro LAN light assembly instructions</b>	5
<b>Activating the license key</b>	7
<b>Using the qubeAV application</b>	9
Connecting and setting up audio/video	9
Integrating an external medium	13
Setting preroll	13
Marker set	14
Recording	15
Using the tile view	18
Saving a recording	19
Opening a saved recording	20
Processing recordings further	20
Closing the application	21
Shortcuts in the software	22
Marker shortcuts	23
<b>Disposing of the qubeAVpro LAN light system</b>	24
<b>Legal notice</b>	24



## General

### Introduction

Congratulations on your purchase of the new qubeAVpro LAN light.

With the current interest in optimizing training in team communication and medical procedures, the use of audio-video systems has become a hugely popular part of regular training.

You can use your debriefing system to analyze your training scenarios from all angles. The high-quality qubeAV software allows you to record these training scenarios for later debriefing.

The qubeAV software lets you record a patient monitor, which makes it easy for you to integrate systems from the qubeSERIES into the qubeAVpro LAN light system in a simulation scenario.

Markers let the instructor mark important scenarios in the simulation at any time to ensure even quicker access to important analyzes during the debriefing.

### Sustainable quality management

The SKILLQUBE “Mission360” relies upon holistic solutions for simulations and training in preclinical and clinical patient care. For SKILLQUBE, “Mission360” means sustainably supporting every area of learning through providing significant input.



Its scope encompasses simple knowledge transfers, teaching new skills, training people to make the right decisions, team simulations and training sessions, conveying preclinical and clinical experiences and everything in between.

With the qubeAVpro LAN light audio-video system, SKILLQUBE has revolutionized the analysis and debriefing of simulation scenarios.

This manual provides you not only with the operating instructions but also with an abundance of information about working with the qubeAV software. To ensure that you enjoy using your qubeAVpro LAN light system for a long time to come, please take time to carefully read through the instructions and safety information below.



## **Important safety measures**

At SKILLQUBE, safety comes first!

We keep the safety of our customers in mind at all times when designing and assembling our products.

### **Safety measures for your qubeAVpro LAN light system**

- Never leave the qubeAVpro LAN light system operating unattended.
- Make sure that the system has been assembled correctly and completely prior to commissioning.
- Never allow water or other fluids to enter the system.
- Always remove the power plug from the power outlet when you have finished working with the qubeAVpro LAN light system.
- Never pull or yank on the power cable.
- Read through the instructions carefully before putting the device into operation. Keep the instructions accessible at all times.
- Never immerse the power cable in water or other liquids.
- Never work with the system if the power cable or plug is damaged or if the system is not working as expected or is in some way damaged. For maintenance or repair, take the device only to an authorized person or return it to SKILLQUBE.
- Never let the power cable hang over the edge of the table. Keep the power cable away from hot surfaces (for example, radiators and stoves).
- Always turn off the system and unplug the power plug from the power outlet:
  - If you leave the system unattended
  - If the system is not in operation
  - Before cleaning the system
  - Before moving or transporting the system
- All maintenance work beyond the cleaning of the exterior of the qubeAVpro LAN light system must be carried out only by personnel authorized by SKILLQUBE.

## qubeAVpro LAN light assembly instructions

The qubeAVpro LAN light debriefing system consists of:

- Box with rack case
  - Connections for accessories
  - MacBook
- Boxes with accessories
  - 3x 20 m LAN cable
  - 3x IP camera box
  - 1x speaker including power supply
  - 1x 10 m audio cable
  - 1x desktop microphone
  - 1x preamplifier including power pack
  - 1x rack case power cable
  - 1x MacBook power supply
  - 1x USB marker keyboard



### Rack case connections:




From left to right:

- On/off switch
- HDMI in: external media stream (optional)
- LAN: connection for IP cameras
- WAN: external maintenance
- USB: connection for marker keyboard
- HDMI out: auditorium monitor
- Power Twist socket: power supply

**qubeAVpro LAN light “Voice of God” (communication station)**

You will receive different preamplifiers depending on their availability, but they are all identical with regard to their “Voice of God” function.

1. Connect the microphone (6.3 mm jack) to the preamplifier (“Mic in”).
2. Connect the preamplifier (red/white RCA connectors) to the speaker (3.5 mm jack – “AUX in”).
3. Connect the preamplifier and speaker to the power supply.

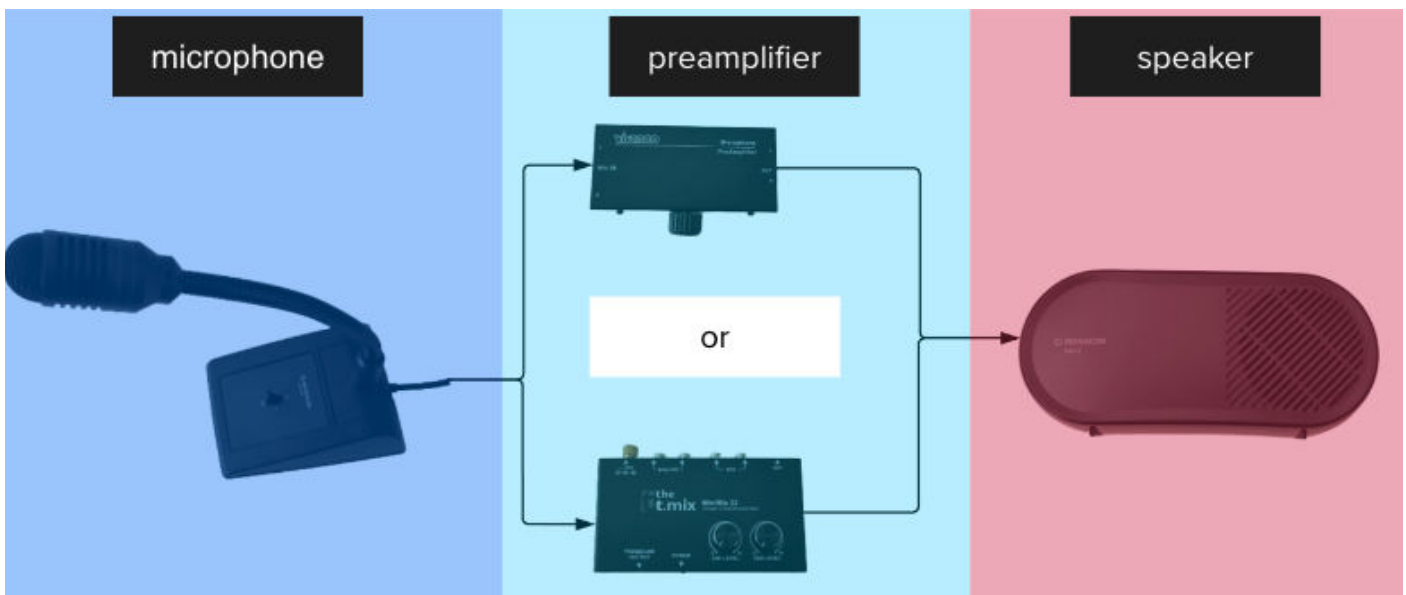


## Activating the license key

The qubeAVpro LAN light debriefing system that you receive is usually already fully set up. If this is the case, you can skip this point. If a license key has not been entered yet, proceed as follows:

1. Open the installed qubeAV application in the dock.




2. After opening the application, you land on the start screen. To activate the license, press the “Open Settings” button.

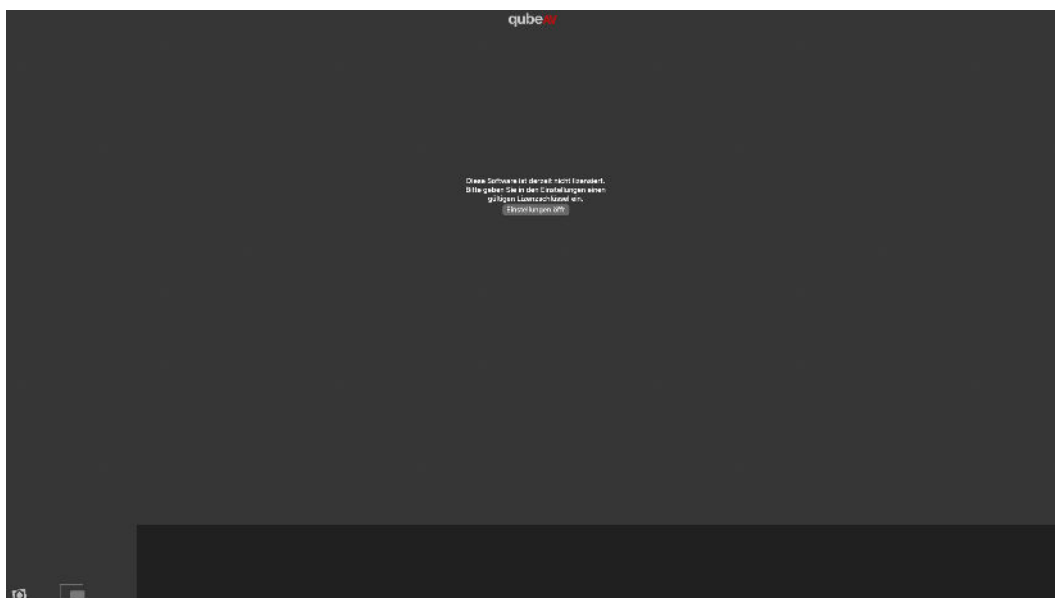




3. Open the lock in the bottom left corner. You can then enter the license key that was sent to you.

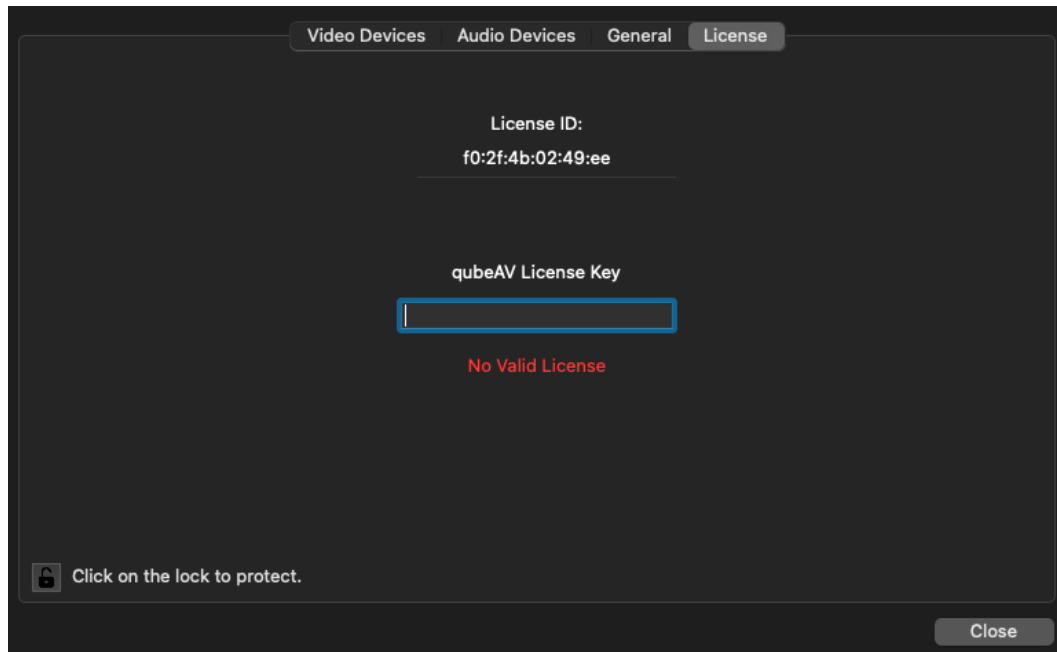

4. The application starts itself up. You do not have to provide any confirmation after entering the correct license key. After entering the key, close the lock in the bottom left corner and then choose "Close".



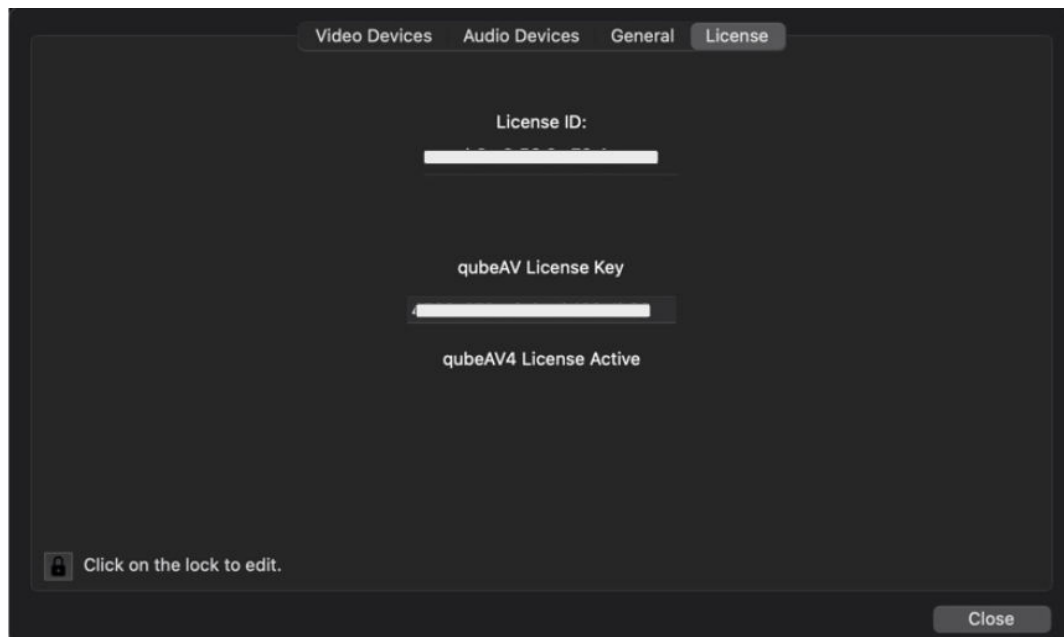

## Using the qubeAV application

### Connecting and setting up audio/video


1. Open the installed qubeAV application in the dock.




2. If no cameras are found after you open the software, you can go directly to the settings to configure them. If you just want to load a file to view it, you can skip this step.




3. Use the shortcut  $\text{⌘} +$  or go to “qubeAV” and then “Settings” in the menu bar at the top to open the settings.

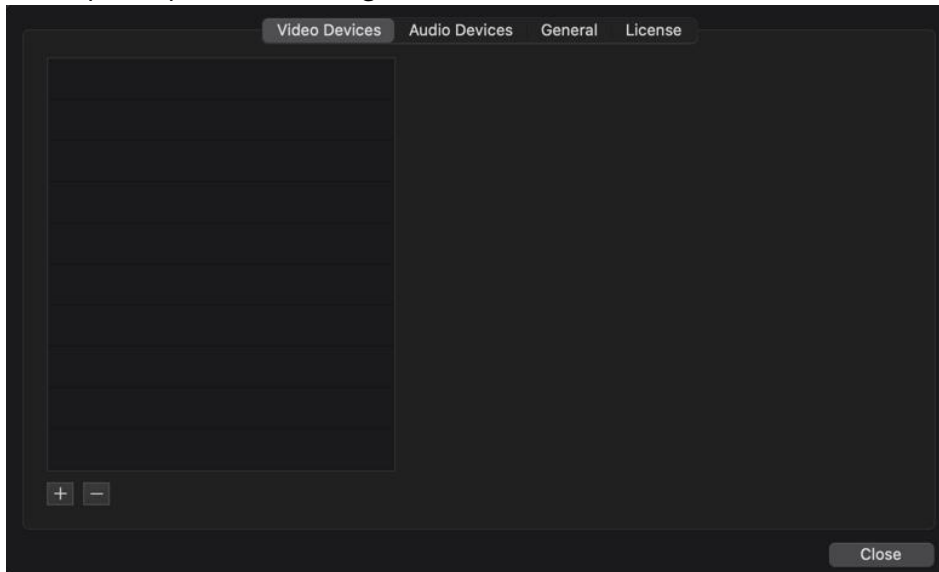


4. Press “+” in the bottom left corner to open the connected cameras. All the available devices are displayed for you. If you are using a qube patient monitor, you can also integrate it into the system here. The monitor is listed by name if it is on the same network as your debriefing system and has been started.

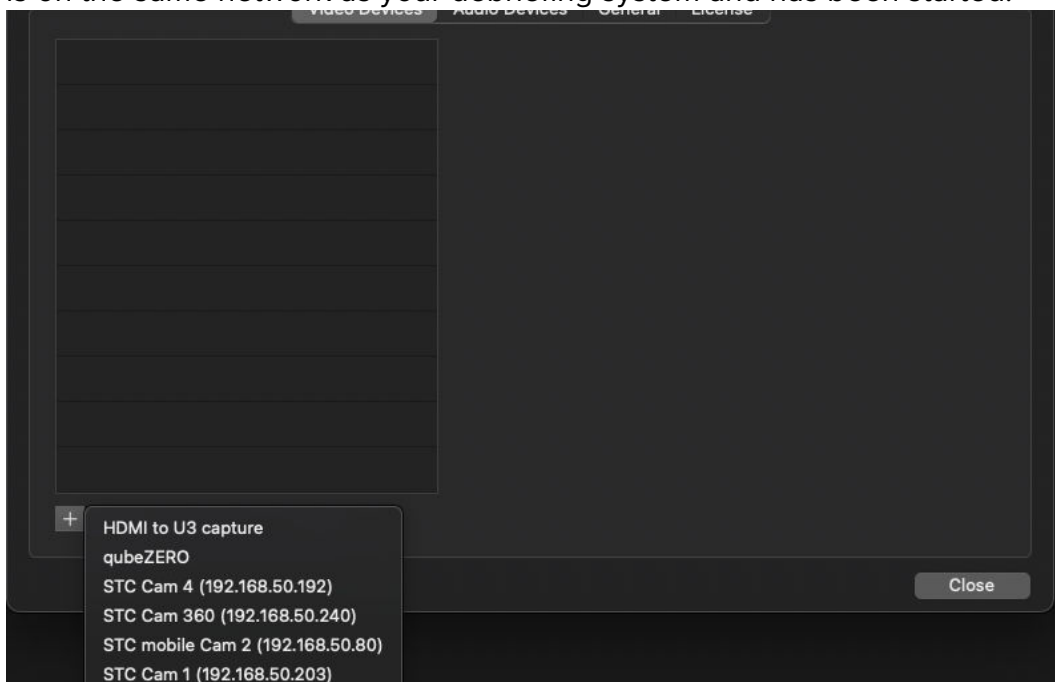


5. Once you have clicked the desired camera to select it, it appears in your list. Repeat this step to connect any remaining video sources.

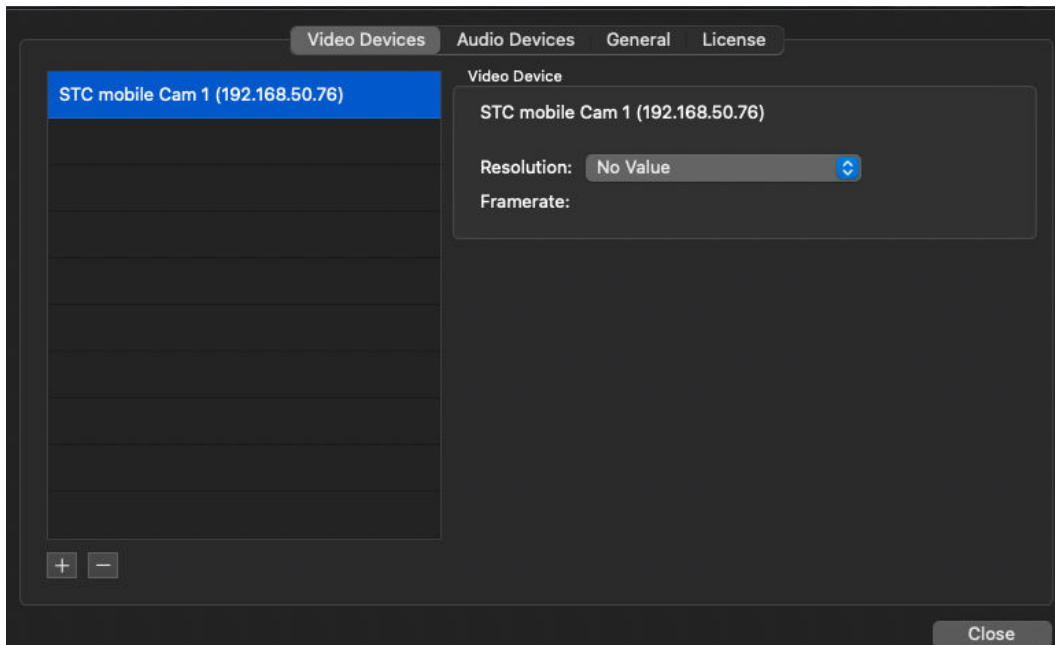

6. You can now select the device that you want to provide you with the audio signal under “Audio Devices” in the menu. Press “+” in the bottom left corner to display all the available devices.

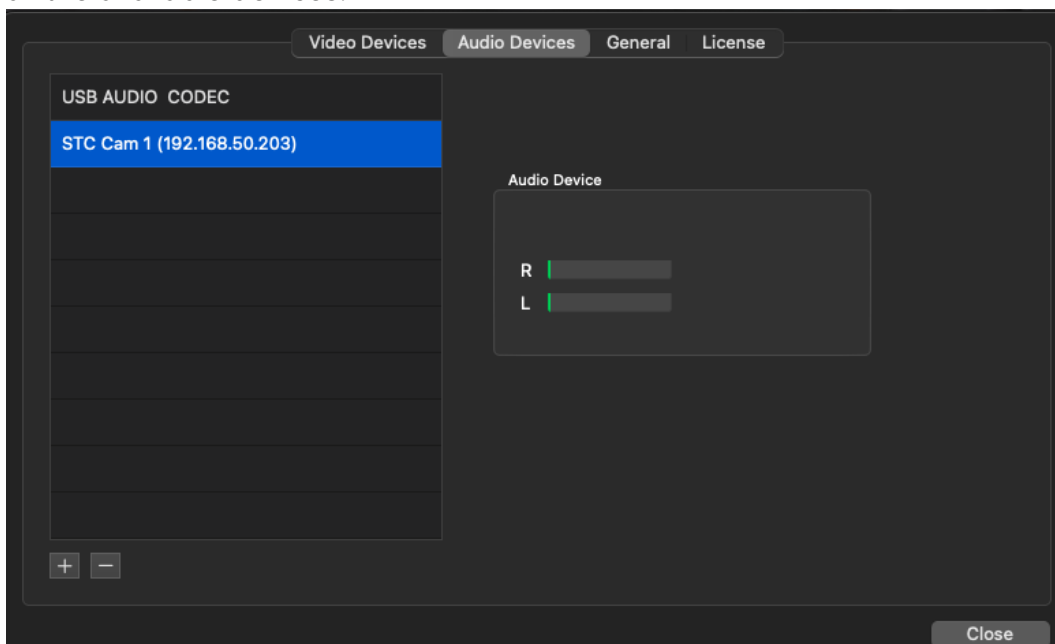

7. Create a background noise in the simulation room to check that the microphone is working. If the microphone works, the bars in R and L move out.

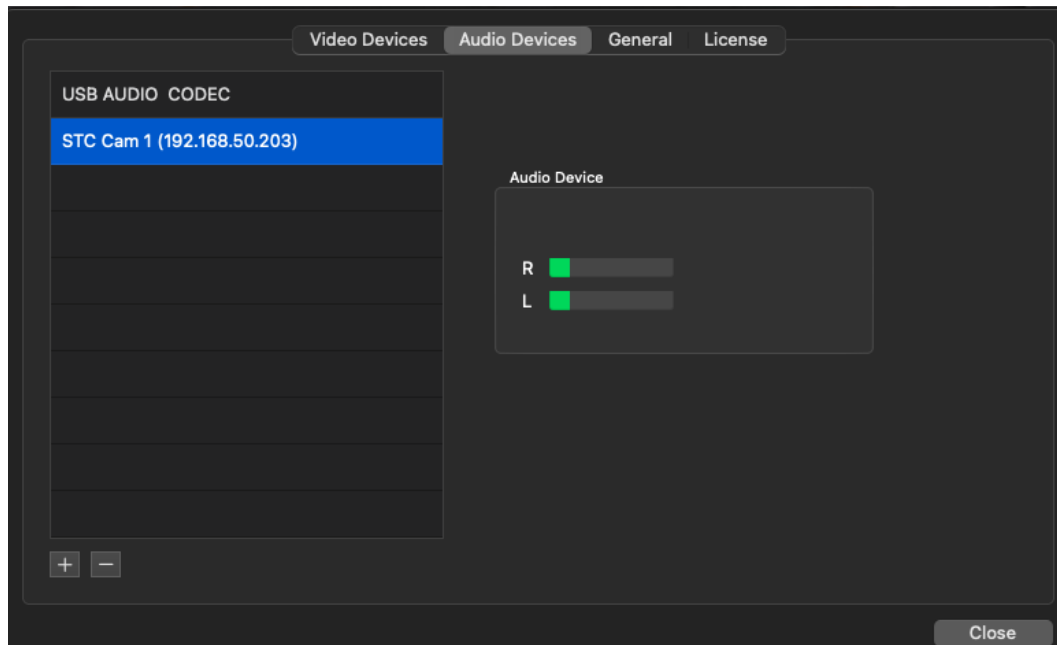

8. Once you finish configuring the settings, press the “Close” button.

## Integrating an external medium

Proceed as you did when setting up the cameras.

1. Settings
2. Video sources
3. Press “+” in the bottom left corner.

Then select the source “HDMI to U3 capture”.

You can then add this source to the tile view and access it there.


**Important:** Only connect the external medium if a signal is present on the HDMI input (“HMDI in”).

## Setting preroll

Preroll is a way to control the time at which a marker is set. If you do not use preroll, the marker is set at exactly the time of the click. As a result, however, the marker may not be positioned at the beginning of the relevant situation, but a little later. To compensate for our reaction and response times, you can go to the “Preroll” submenu and configure the marker so that it is automatically set a few seconds before the click.

To open the “Preroll” submenu, proceed as follows:

1. In the menu bar at the top of the screen, click “qubeAV”.
2. In the submenu that opens, click “Settings”.
3. On the “Settings” submenu, click the “General” tab.
4. Here, you can set a preroll of up to five seconds before the click.

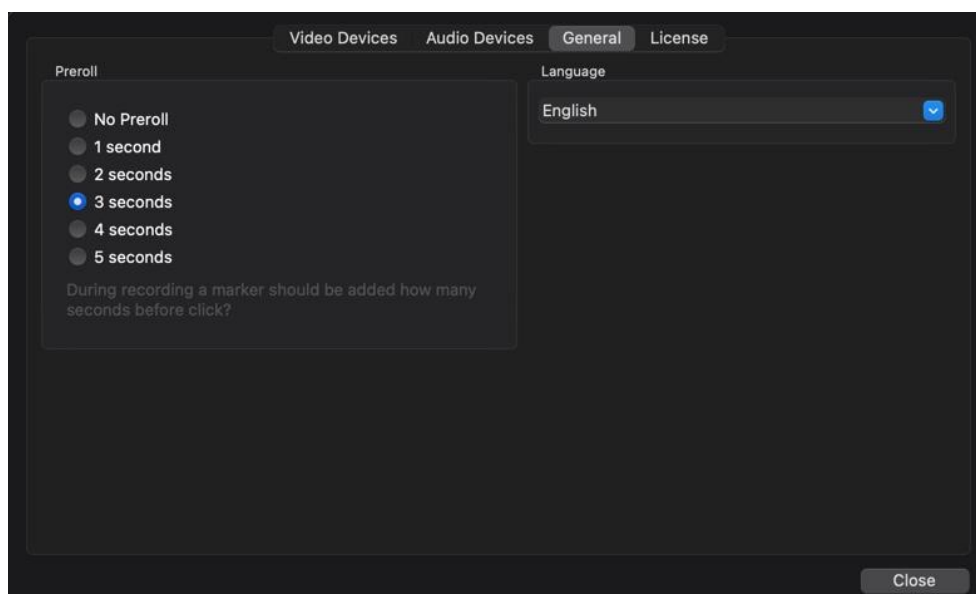




## Marker set

Eight predefined markers are stored in your software. These markers were configured by SKILLQUBE and synchronized with the marker keyboard provided. The markers cannot be edited.



## Recording

1. To enable viewers to see an image on the external monitor as well, click the monitor icon in the bottom left corner to activate the extended display (student view). Audio is enabled automatically. You can adjust the volume using the debriefing volume slider.
2. Press the red record button to start recording. When you start recording, the button changes to a stop symbol.

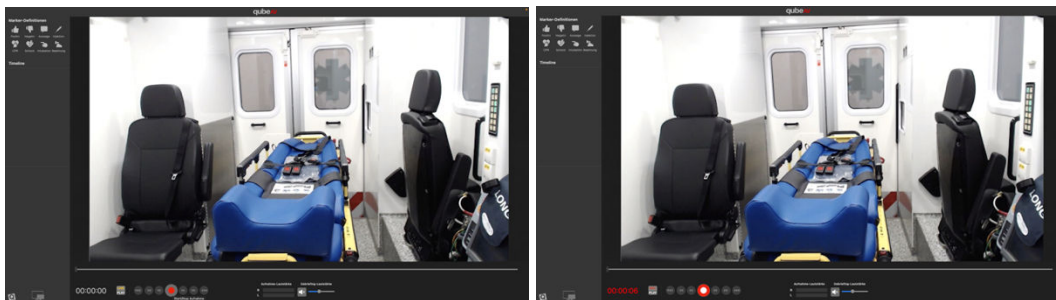

3. To finish recording, press the red button again and confirm that you want to finish recording in the dialog menu that appears. The view switches to replay mode and you can begin the debriefing. The image displayed is blurred until you begin the debriefing.



## Playing a recording



1. Enable the extended display (student view)
2. Recording control panel: From left to right:
  - a. Previous marker
  - b. Go to the start of the recording
  - c. Go five seconds back
  - d. Play/pause replay
  - e. Go five seconds forward
  - f. Go to the end of the recording
  - g. Next marker
3. Jump to: enter a time in the recording
4. Volume control
5. Time of the current position in the recording, with the total length of the recording below that
6. Summary of the set markers, with time stamps and comments
7. Go to the live view






When you pause the video in replay mode, the current displayed image is provided with a blur effect.

This effect is removed automatically when you continue playback. If you require the current image, click the view once and the filter is removed.



## Using the tile view

You have the option of enlarging an individual window during recording. Simply click the desired image to do so. .

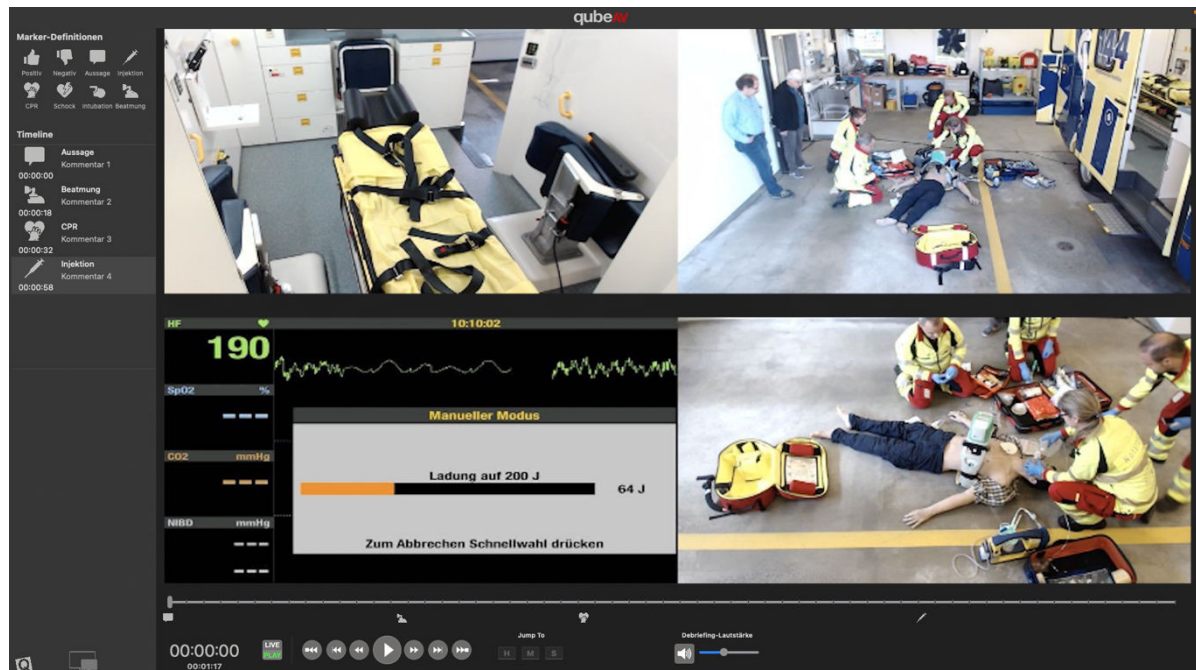

You can also enlarge the image in the view provided. To do so, click the area that you wish to enlarge. Click it again to zoom out.



To return to the tile view, click the white grid symbol in the top right corner. Symbol: white grid with four panels



In recording mode, only the instructor can go into the views; the auditorium continues to see the tile views.

During debriefing, you can make individual images available to the auditorium and zoom into the video in this way.

### **Saving a recording**

1. When you switch to the live view, the system asks whether you want to save the recording.
2. Press the “Save recording” button and select the storage location for your file.


## Opening a saved recording

Use the shortcut  $\text{⌘} + \text{o}$  or go to “File” and “Load Video” in the menu bar. Select the video that you want to open and press the “Open” button.



## Processing recordings further

The recordings are saved in “.qavpro” format. If you want to continue processing the recordings in third-party programs, you can access the individual video tracks and the audio track by right-clicking the file and choosing “Show Package”.

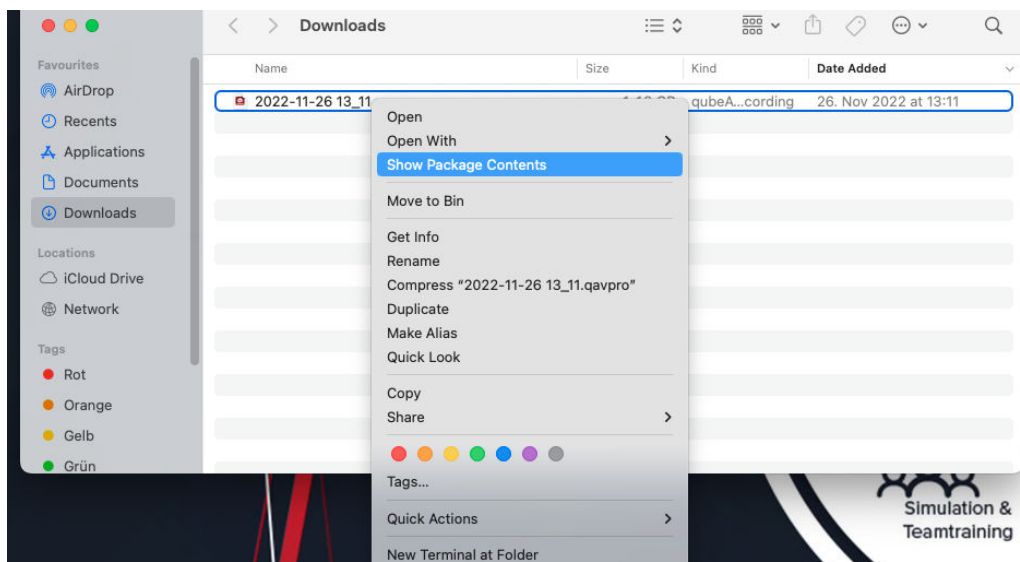




## Closing the application

Whenever you attempt to close the application while working, the system asks whether you really want to close the application.

To close it, use the shortcut **⌘ + X** or press the red button in the top left corner.

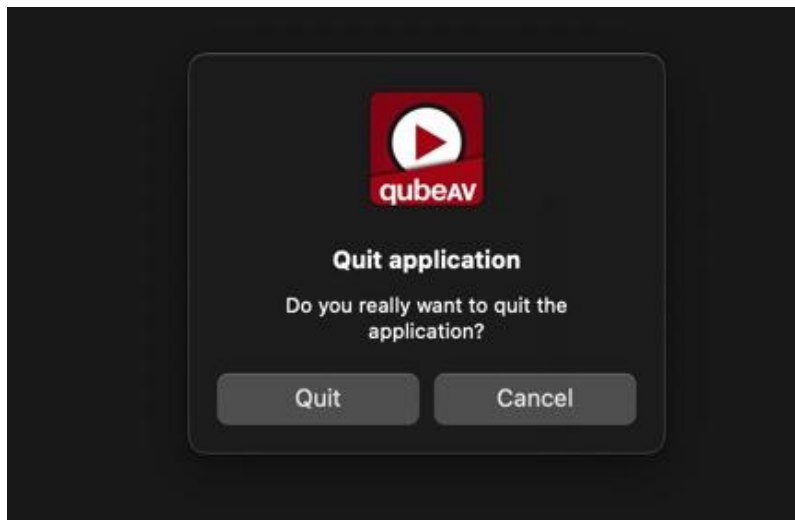

## Shortcuts in the software

The term “shortcut” refers to the act of pressing one or more specific keys on a keyboard individually or at the same time. This causes the qubeAV software to execute the commands that are assigned to this shortcut.

Benefit:

Above all, shortcuts save time. Shortcut keys allow you to execute many commands much faster than with the mouse.

Keyboard shortcut	Action
⌘ + ,	Opens the settings
x	Closes the current action
e	Activate/deactivate the auditorium
a	Turn audio on/off
⌘ + +	Audio volume up
⌘ + -	Audio volume down
r	Start/stop the recording
p in play mode	Start/stop the replay of the recording
▶ in play mode	Jump to the next marker in the time bar
◀ in play mode	Jump to the previous marker in the time bar
⌘ + ⏮ in play mode	Jump five seconds forward
⌘ + ⏪ in play mode	Jump five seconds backward
⌘ + ▶	Fast-forward in five-second steps
⌘ + ◀	Rewind in five-second steps
⌘ + ▶	Go to the end of the recording
⌘ + ◀	Go to the start of the recording
l	Switch to the live view
⌘ + o	Open a saved recording
⌘ + q	Close the qubeAV software



## Marker shortcuts

Keyboard shortcut	Action
↑ + 1	Positive marker
↑ + 2	Negative marker
↑ + 3	Statement marker
↑ + 4	Injection marker
↑ + 5	CPR marker
↑ + 6	Shock marker
↑ + 7	Intubation marker
↑ + 8	Ventilation marker



## Disposing of the qubeAVpro LAN light system

The qubeAVpro LAN light debriefing system contains a variety of valuable raw materials that can be recycled. At the same time, the devices contain hazardous substances that have to be removed in a way that ensures humans and the environment are not adversely affected.

SKILLQUBE guarantees that all the debriefing systems sent to us for disposal are disposed of and recycled in an environmentally friendly way.

## Legal notice

SKILLQUBE GmbH  
Schafäckerweg 24, 69168 Wiesloch, Germany

District Court – Registration Court – Mannheim HRB 716968  
Head Office: Wiesloch – Managing director: Julia Wallburg, authorized signatories:  
Georg Hirsch, Thorben Schreyer

E-mail: info@skillqube.com  
Homepage: www.skillqube.com

Phone: +49 (0) 6222 / 38 62 65  
Fax.: +49 (0) 6222 / 38 63 65

VAT ID no. DE 289937847

You can view our general terms and conditions at any time at  
<https://en.skillqube.com/legal-notice/>.